

- Real-Time Image & Report Access
- Simplified Patient Search
- Enhanced Image Viewer Tool Options
- Side-by-Side Comparison

Account Access

Required URL Access

myihsonline.com/ica

Recommended Browsers

 Google Chrome &  Internet Explorer

- Notes:
1. Safari is not a Supported Browser
 2. Printing images requires IE browser use

Login


Enter assigned User ID and Password (Case Sensitive)




Password Change

For security, IHS recommends you change your password on your first log in. To change your password:

OPTIONS > MyProfile > Enter and Confirm New Password > SAVE

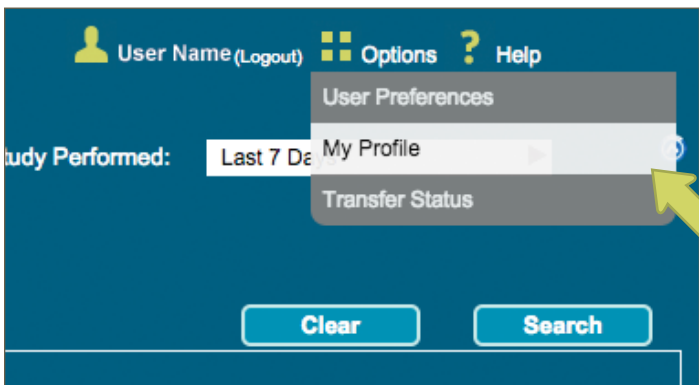

CHOOSE 'MY PROFILE' FROM 'OPTIONS' DROP-DOWN MENU

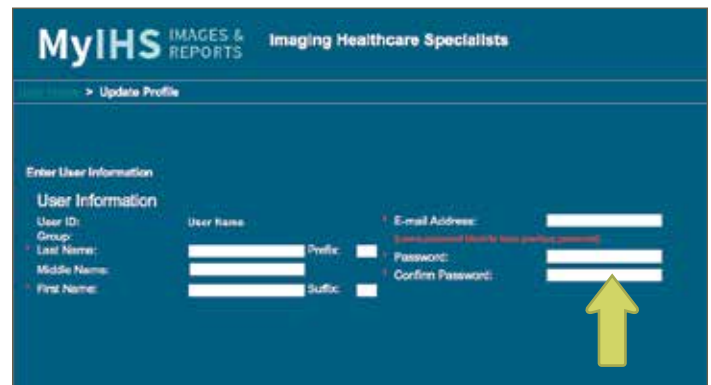

ENTER AND CONFIRM NEW PASSWORD AND CLICK 'SAVE'

View Images & Reports

There are two options to view patients. Patient exams will populate to work list below search fields.

Option 1 Enter Last Name and First Name of the patient & Click “Search”

Date of Birth is recommended only for common name searches

Option 2 Click radio button for: “My Patients Only” & Click “Search”

A Work list of **only user referred patients** will populate per **study performed time period** identified


The screenshot shows the MyIHS IMAGES & REPORTS search interface. At the top, it says "MyIHS IMAGES & REPORTS Imaging Healthcare Specialists". There are user and help icons in the top right. The search form includes: Last Name, First Name, Date of Birth, Modality, Ref. Physician (radio button), My Patients Only (radio button), and Study Performed (Last 7 Days). Below these are buttons for My Search, Save, Delete, Clear, and Search. At the bottom, there is a "Group By" dropdown menu set to "No Grouping" and a table header with columns: Study Date, Modality, Patient Name, Patient DOB, Patient ID, Description, and Ref. Physician.

Navigating Image Viewer

Search Window Options

- **STUDY PERFORMED:** Drop-down menu allowing user to choose date range for patient exams. Any exam search will follow the time frame choice in this field.
- **MODALITY:** Not required.
- **REF PHYSICIAN:** Radio button identifies you are searching for a patient exam outside of your referral/practice. Not required.
- **MY PATIENTS ONLY:** Populates only patient exams performed with your name as the referrer or where you were noted as a “cc” on the dictated report.

Image Viewing

- **OPEN IMAGE VIEWER:** Double click on your selected/highlighted exam
- **IMAGE TO FULL SCREEN:** Click F11 function key to bring single image to full screen. Click again to return image to original browser settings
- **TOOL USE – FULL SCREEN:** Click  to enlarge currently viewed images and hide “thumbnails and toolbar”.
- **MENUS AND HISTORY TABS:** Located to the LEFT MARGIN. Click **MENUS** to view/hide toolbar and thumbnail series boxes. Click **HISTORY** to view patient exam history (verify DOB for every exam).
- **THUMBNAILS:** Small image windows appear across top of Image Viewer screen. Each window represents first image of series for complex imaging; or “each” static image (e.g., x-ray, ultrasound, mammography). Drag and Drop Thumbnails to any window for viewing.

Report Viewing

- Report Icon:  “Yellow Notebook Page” Icon – Click to open exam report